

Discover... savvy!

STARLIGHT EXPRESS®

Clear button and clear-aligner button former with light-polymerisation in pencil-size




This small light-cure pen forms a tooth-colored button quickly and economically.

Simply fill the preformed tip with your light-curing bonding or resin material in syringes and light-cure for a button that is virtually invisible.



- Compact pencil size
- Light weight

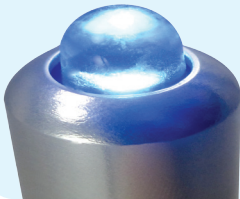

- Battery powered for convenience
- Fast and easy to use



- Clear button former
- Clear clear-aligner button former
- Can be also used on most standard light-curing lamps




- Fast curing time with approx.. 30-50 sec
- Economical



Bringing German Engineering to Orthodontics

STARLIGHT EXPRESS

User Instructions

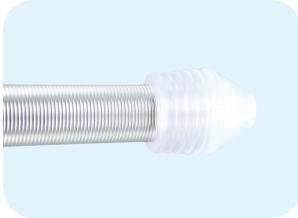

Simply push the button former on the top of the STARLIGHT EXPRESS.




Fill the preformed button former with your light-curing bonding or resin material in a syringe.



Etch the tooth as usual.



Press the STARLIGHT EXPRESS lightly on the relevant tooth and cure according to the bonding/resin material instructions, approx. 30-40 sec

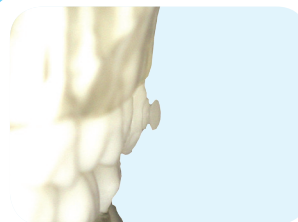

After curing remove the STARLIGHT EXPRESS carefully with a small tipping motion and reveal your tooth colored button bonded and ready to use.



Bringing German Engineering to Orthodontics